

## What's Included



HTC XR Elite Headset (1x)



HTC XR Elite Detachable Head Strap Battery (1x)



Detachable Face Gasket (3x)  
(will arrive connected to the headset)



Care Tablet with Case (1x)  
(will arrive with case attached)



Auxiliary Battery with Velcro Case (1x)



USB-C Charging Cords (4x)



100w USB-C Charging Brick (1x)  
(do not use a standard 8w charging brick)



Mynd Cleaning Cloth (1x)



Hard Case with Custom Foam Insert (1x)



HTC Elite Hand Controllers (2x)

## Charging the System Batteries

The USB-C charging cables and the 100W USB-C charging brick are used to charge all the hardware supplied, including:

**Auxiliary Battery:** To charge the Auxiliary Battery, connect it to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet. You can check the Battery's charge by pressing the small black button on the side of the Battery. A fully charged Auxiliary Battery will power 2-3 hours of consistent Headset use.

**Head Strap Battery:** To charge the Detachable Head Strap Battery, connect it to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet. A fully charged Head Strap Battery will power 1-2 hours of consistent Headset use.

**Care Tablet:** Connect the Tablet to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet.

**Hand Controllers:** Connect the Hand Controllers to the charging brick using the USB-C charging cords and plug the charging brick into a standard wall outlet.

**Headset:** The HTC XR Elite Headset has an internal battery that typically does not need to be charged. However, if necessary, connect the Headset to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet.

## First-Time Setup and Connecting to the Internet

**Hotspot (optional):** If you received a Mynd-approved hotspot, both the Tablet and Headset will be pre-configured to connect to the hotspot network. The hotspot requires no additional setup. Simply power on the hotspot, and it will start broadcasting a signal. Then proceed to the "Using Devices After First-Time Setup" section. If you did not receive a hotspot, follow the steps below to get connected to your network.

**Tablet:** To power up the Tablet, press and hold the power button on the left side of the Tablet for 2 seconds. The Tablet will enter a power-up sequence. After 30-40 seconds, the Tablet will display its home screen, with the Mynd application in the center. Swipe up on the Tablet screen to access all applications and select "Settings." Within Settings, select "Network and Internet" and then "Wi-Fi." Select your desired network and enter the password. Once connected, exit the Settings menu by either tapping the small circle at the bottom of the screen or swiping up from the bottom, depending on your Tablet model. Select the Mynd application, press "Start," and navigate through the setup wizard. Once completed, the screen will display a message that says "Searching for Headsets."

**Headset:** To power up the Headset, connect either the Head Strap Battery or the Auxiliary Battery to the USB-C connection port on the right-hand frame of the Headset.

The Headset will automatically enter a power-up sequence in which you might see different images and menus. If it does not automatically start, press the small circular button on the top left of the Headset. After 30 seconds, the Headset will automatically enter the Mynd Headset application. Here, you should see a beach setting with a graphic that says "Waiting for Tablet to select app..."

To connect the Headset to the internet, first grab the charged right-hand Controller. If it's powered off, press the small triangle button once to turn it on. The Controller will vibrate and display a green light on the front, indicating that it's paired with the Headset. Once the Controller is powered on and connected, press the triangle button again to bring up the pop-up menu. From the pop-up menu, make sure "Recent" is selected from the bottom banner. Then, use the trigger button on the back of the Controller to select "Quit," and then "Quit App." After you quit the application, the Headset will display the lobby screen. Once in the lobby, select "Settings" from the bottom banner, select "Connectivity" on the left side of the menu, and then choose "Wi-Fi." Connect to the same network that you used for the Tablet. If you don't see any networks displayed, check to ensure the Wi-Fi toggle button is toggled on. You can use the same trigger button on the Controller to type your password. After connecting, select "Library" from the bottom banner, then select the "Mynd Launcher" application to return to the beach setting.

After the Headset connects to the network and you return to the beach setting, the Tablet will automatically detect the Headset and display it in the Headset Management banner located in the bottom-left corner of the Tablet screen. This process may take up to 15 seconds. Detected Headsets will appear as gray or green squares labeled with their serial numbers. A green square indicates that the Headset is active and can be controlled from the Tablet. If the Headset does not appear on the Headset Management banner, restart the Headset and then proceed to the next section, "Using Devices After First-Time Setup."

## Using Devices After First-Time Setup

If the Tablet and Headset have undergone the initial setup process, they should automatically connect to your network every time they are powered on. Follow the steps below to begin using the system. **Important:** Always start the Tablet application before powering up the Headset.

**Hotspot (optional):** If you received a Mynd-approved hotspot, begin by powering up the hotspot, and it will start broadcasting a signal.

**Tablet:** To power up the Tablet, press and hold the power button on the left side of the Tablet for 2 seconds. The Tablet will enter a power-up sequence. After 30-40 seconds, the Tablet will display its home screen, with the Mynd application in the center. Select the Mynd application, press "Start," and navigate through the setup wizard. Once completed, the screen will display a message that says "Searching for Headsets."

**Headset:** To power up the Headset, connect either the Head Strap Battery or the Auxillary Battery to the USB-C connection port on the right-hand frame of the Headset. The Headset will automatically enter a power-up sequence in which you might see different images and menus. If it does not automatically start, press the small circular button on the top left of the Headset. After 30 seconds, the Headset will enter the Mynd Headset application. Here, you should see a beach setting with a graphic that says "Waiting for Tablet to select app..." Once in the beach setting, the Headset will appear in the Headset Management banner on the Tablet, allowing you to start using the system.

## Managing Multiple Headsets and Tablets

The Headset Management banner on the Tablet enables you to manage multiple Headsets and Tablets simultaneously.

If attempting to use multiple Headsets, power on each Headset as per the setup instructions above. Detected Headsets will appear on the Tablet as gray or green squares labeled with their serial numbers. A green square indicates that the Headset is active and can be controlled from the Tablet. A gray square indicates an inactive Headset. Headsets can be identified by the serial number located on the top of the device. To control an inactive Headset, simply tap the corresponding gray square. The square will turn green, and you will now be in control of that Headset. Keep in mind, only one Headset can be active at a time.

If multiple Tablets are powered on, the Headset Management button on each Tablet can be used to choose which Headsets to control.

## Controlling the Headset from the Tablet

**Mynd Applications button:** When you tap the "Mynd Applications" button, a menu will appear showing all available applications. From this menu, the Tablet user can launch any application.

**Reorient View button:** Tapping the "Reorient View" button resets the Headset's display to a level forward-facing position. This is useful for recalibrating the view if it becomes tilted or for adjusting orientation when a user changes position, such as lying down.

## Powering Down/Restarting Headsets and Tablets

**Tablet:** Press and hold the power button for 1-2 seconds, then select either "Power Off" or "Restart."

**Headset:** Press and hold the small circular button on the top left of the Headset for 1-2 seconds, then select either "Shutdown" or "Restart Device."



Scan the QR code or visit [www.MyndImmersive.com/quickstart](http://www.MyndImmersive.com/quickstart) for support resources designed to help you get the most out of your Mynd system.