

## What's Included



HTC Flow Headset (1x)



Detachable Face Gasket (2x)  
(will arrive connected to the headset)



HTC Flow Hand Controller (1x)



Care Tablet with Case (1x)  
(will arrive with case attached)



Auxiliary Battery (1x)



USB-C Charging Cords (2x)



100w Charging Brick (1x)  
(do not use a standard 8w charging brick)



Mynd Cleaning Cloth (1x)



HTC Flow Protective Pouch (1x)



HTC Flow Headstrap (1x)



Hard Case with Custom Foam Insert (1x)

## Charging the System Batteries

The USB-C charging cables and the 100W USB-C charging brick are used to charge all the hardware supplied, including:

**Auxiliary Battery:** To charge the Auxiliary Battery, connect it to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet. You can check the Battery's charge by pressing the small black button on the side of the Battery. A fully charged Battery will power 2-3 hours of consistent Headset use.

**Care Tablet:** Connect the Tablet to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet.


**Headset:** The HTC Flow Headset has an internal battery that typically does not need to be charged. However, if necessary, connect the Headset to the charging brick using a USB-C charging cord and plug the charging brick into a standard wall outlet.

## First-Time Setup and Connecting to the Internet

**Hotspot (optional):** If you received a Mynd-approved hotspot, both the Tablet and Headset will be pre-configured to connect to the hotspot network. The hotspot requires no additional setup. Simply power on the hotspot, and it will start broadcasting a signal. Then proceed to the "Using Devices After First-Time Setup" section. If you did not receive a hotspot, follow the steps below to get connected to your network.

**Tablet:** To power up the Tablet, press and hold the power button on the left side of the Tablet for 2 seconds. The Tablet will enter a power-up sequence. After 30-40 seconds, the Tablet will display its home screen with the Mynd application in the center. Swipe up on the Tablet screen to access all applications and select "Settings." Within Settings, select "Network and Internet" and then "Wi-Fi". Select your desired network and enter the password. Once connected, exit the Settings menu by either tapping the small circle at the bottom of the screen or swiping up from the bottom, depending on your Tablet model. This will take you back to the home screen. Select the Mynd application, press "Start," and navigate through the setup wizard. Once completed, you will see a screen displaying "Searching for Headsets."

**Headset:** To power up the Headset, attach a USB-C cord to the (charged) Auxiliary Battery and connect the other end to the right-hand frame of the Headset. See reference image to the right.

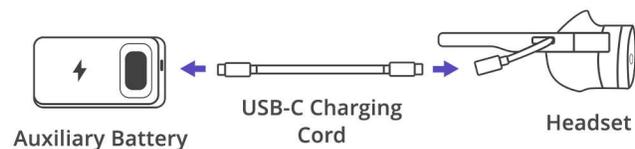

The Headset will automatically enter a power-up sequence in which you might see different images and menus. If it does not automatically start, press the small circular button on the top-right of the of the Headset. After 30 seconds, the Headset will transition to the Library Menu, where you will see various applications and icons.

To connect the Headset to the internet, begin by picking up the Hand Controller. Press and hold the small circular button at the bottom of the Controller. Once pressed, the Controller will vibrate and pair with the Headset. Press and hold the same button again to display the Controller's laser in the Headset. Use the large button at the top of the Controller to select "Settings" from the bottom banner. Select "Connectivity," then select "Wi-Fi." Connect to the same network that you used for the Tablet. If you don't see any networks displayed, check to ensure the Wi-Fi toggle button is toggled on. Use the large button at the top of the Controller to type your password. Once connected, restart the Headset by holding down the small circular button on the top-right of the Headset for 2 seconds. Select "Restart Device."

After the Headset restarts, use the Controller to select "Mynd Launcher" from the Library Menu. The Headset will automatically launch the Mynd Headset application, where you will see a beach setting with a graphic displaying the message: "Waiting for Tablet to select app..." The Tablet will automatically detect nearby Headsets and display them in the Headset Management banner at the bottom left of the Tablet screen. This process may take up to 15 seconds. Detected Headsets will appear as gray or green squares displaying their serial numbers. A green square indicates that the Headset is already active.

## Using Devices After First-Time Setup

If the Tablet and Headset have undergone the initial setup process, they should automatically connect to your network every time they are powered on. Follow the steps below to begin using the system. **Important:** Always start the Tablet application before powering up the Headset.

**Hotspot (optional):** If you received a Mynd-approved hotspot, begin by powering up the hotspot, and it will start broadcasting a signal.

**Tablet:** To power up the Tablet, press and hold the power button on the left side of the Tablet for 2 seconds. The Tablet will enter a power-up sequence. After 30-40 seconds, the Tablet will display its home screen, with the Mynd application in the center. Select the Mynd application, press "Start," and navigate through the setup wizard. Once completed, you will see a screen displaying "Searching for Headsets."

**Headset:** To power up the Headset, attach a USB-C cord to the (charged) Auxiliary Battery and connect the other end to the right-hand frame of the Headset. If it does not automatically start, press the small circular button on the top-right of the Headset. After 30 seconds, the Headset will transition to the Library Menu. Select "Mynd Launcher" from the menu to launch the Mynd Headset application. Once opened, you will see a beach setting with a graphic displaying the message: "Waiting for Tablet to select app..." Depending on your version, the Headset may bypass the Library Menu and go directly to the beach setting when powered up. When the beach setting appears, the Headset will show up in the Headset Management banner on the Tablet, allowing you to start using the system.

## Managing Multiple Headsets and Tablets

The Headset Management banner on the Tablet enables you to manage multiple Headsets and Tablets simultaneously.

If attempting to use multiple Headsets, power on each Headset as per the setup instructions above. Detected Headsets will appear on the Tablet as gray or green squares displaying their serial numbers. A green square indicates that the Headset is already active. A gray square indicates an inactive Headset. Headsets can be identified by the serial number located on the side of the device. To control an inactive Headset, simply tap the corresponding gray square. The square will turn green, and you will now be in control of that Headset. Keep in mind, only one Headset can be active at a time.

If multiple Tablets are powered on, the Headset Management button on each Tablet can be used to choose which Headsets to control.

## Controlling the Headset from the Tablet

**Applications button:** Tap the "Mynd Applications" button to access the list of available apps. After selecting an app, it will launch on both the Tablet and the Headset simultaneously.

**Reorient View button:** Tapping the "Reorient View" button resets the Headset's view to a level field. It can be used to recalibrate the view if it becomes skewed or to adjust the orientation for a user who has changed positions, such as lying down.

## Powering Down/Restarting Headsets and Tablets

**Tablet:** Press and hold the power button for 1–2 seconds, then select either "Power Off" or "Restart."

**Headset:** To restart the Headset, press and hold the small circular button on the top-right of the device for 1–2 seconds, then select "Restart Device." To power it down, disconnect the USB-C cord that connects the Headset to the Battery.



Scan the QR code or visit [www.MyndImmersive.com/quickstart](http://www.MyndImmersive.com/quickstart) for instant access to valuable support materials that will enhance your experience.